Training Device

Trainhaler is a training device to be used with the Flo-Tone to improve pMDI technique in 3 ways:

- Correct flow
- Co-ordination signal
- Inhalation duration

For multiple patient use

www.clement-clarke.com

HS CLEMENT CLARKE INTERNATIONAL
Engineered for Life

To the Doctor, Nurse or Pharmacist:

Many patients do not use their pMDIs correctly resulting in inadequate intake of medicine.

Trainhaler is for multiple patient use.

Flo-Tone is for individual patient use.

Trainhaler is the first non-electric pMDI training device to provide flow and co-ordination coaching.

Trainhaler is a placebo-like pMDI training device, it contains no medicine.

With the addition of Flo-Tone Trainer, Trainhaler becomes the essential tool for healthcare professionals to teach inhaler technique (flow and co-ordination).

When your patient inhales through the Trainhaler and Flo-Tone Trainer system, you will hear a whistle sound when the patient's flow rate is adequate.

As soon as the whistle starts this is the signal to press the canister (they will hear a noise similar to an actuated pMDI). The patient should keep the whistle going for 5 seconds, if possible. This will ensure that the patient is inhaling gently, at the correct flow rate.

What is this Product used for?

Trainhaler has been designed for use by healthcare professionals to help their patients improve their inhaler technique, resulting in the correct use of pMDIs.

Flo-Tone is for individual patient use to reduce the risk of cross-infection.

Trainhaler should be cleaned between patients.

Storage

- Keep your Trainhaler in a safe place, out of reach from children.
- You should clean your device referring to the 'Cleaning' section.
- Store your Trainhaler below 40° C (104° F)

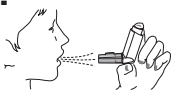
Cleaning

- Clean Trainhaler after every use, following the instructions below.

To clean your Trainhaler:

1. Wipe the inside and outside of the mouth piece recess with a hygienic cloth or tissue.

Instructions for your Patient

1.  Fit Flo-Tone to Trainhaler, creating your unique training system.
2.  Hold the Trainhaler inhaler trainer upright with your thumb on the base, below the mouthpiece. Breathe out as far as is comfortable. Do not breathe in again yet.
3.  Place Flo-Tone between your teeth. Close your lips around it. Do not bite the mouthpiece. Breathe in gently through your mouth.
4.  Just after starting to breathe in, when you hear the Flo-Tone whistle, press down on the top of the canister. You will hear a sound which simulates the puff of aerosol. Continue to breathe in steadily and deeply, keeping the whistle going for as long as you can. (5 seconds).
5.  Hold your breath, remove the inhaler from your mouth. Continue holding your breath for as long as is comfortable. Then breathe normally.

Repeat as necessary to ensure correct technique

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Clement Clarke International Ltd.,