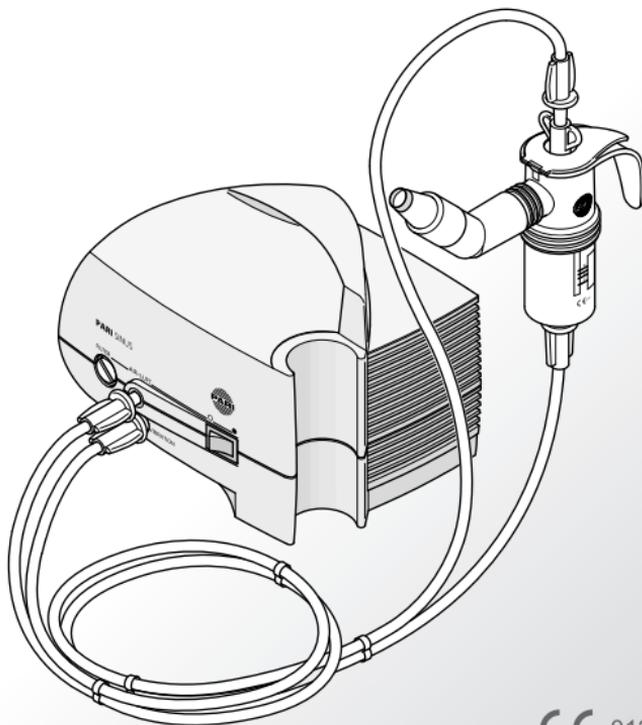
**SINUS**

CE 0123

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**en** **Instructions for use ..... 43**

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# en Instructions for use

Information as of: 2015-06. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

<b>IMPORTANT INFORMATION</b> .....	<b>45</b>
General.....	45
Information about the instructions for use .....	45
Structure of safety instructions .....	45
Treatment of children and anyone who requires assistance .....	46
Using the compressor .....	46
Using the nebuliser.....	47
<b>PRODUCT DESCRIPTION</b> .....	<b>48</b>
Components .....	48
Intended use.....	49
Proper use .....	49
Contraindications.....	50
Material information .....	50
Product combinations .....	50
Operating life .....	51
Working parts .....	52
Mode of action in treatment of the upper airways .....	53
<b>TREATMENT</b> .....	<b>53</b>
Preparing the treatment.....	53
Performing treatment.....	59
Ending the treatment .....	62
<b>CARE</b> .....	<b>63</b>
Preparation .....	63
Drying the connection tubes .....	63
Care of the compressor .....	64
Care of the nebuliser and accessories .....	67

<b>TROUBLESHOOTING .....</b>	<b>72</b>
<b>TECHNICAL DATA.....</b>	<b>73</b>
General compressor data .....	73
Nebuliser data .....	74
Classification according to DIN EN 60601-1 .....	74
Information about electromagnetic compatibility .....	75
Recommended safety distances .....	75
Ambient conditions for operation .....	76
<b>MISCELLANEOUS .....</b>	<b>77</b>
Transportation and storage .....	77
Disposal.....	77
Terms and conditions of warranty .....	77
Explanation of symbols .....	79
Contact .....	80

# 1 IMPORTANT INFORMATION

## 1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

**If symptoms persist or if your condition worsens, discontinue the application and contact your doctor immediately.**

## 1.2 Information about the instructions for use

**These instructions for use are intended for the user at home.**

If these instructions for use are lost, you can request another copy from PARI GmbH [see: Contact, page 80]. The instructions for use for some products can be retrieved on the internet in both German and English. Simply visit:  
[www.pari.de/de-en/products](http://www.pari.de/de-en/products) (on the respective product page).

## 1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

## **1.4 Treatment of children and anyone who requires assistance**

Children and anyone who requires assistance must be supervised constantly by an adult while carrying out the treatment. This is the only way to ensure safe and effective treatment. Individuals in this group often underestimate the hazards involved (e.g. strangulation with the power cord or the connection tubing), thus resulting in a risk of injury.

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

The compressor and the tubes, where they are attached to the compressor, can become very warm during normal operation. Therefore, particularly with individuals who require special assistance, ensure that the user's skin does not come into contact with the compressor. Prolonged contact may cause burns, particularly for patients with impaired temperature perception.

## **1.5 Using the compressor**

The compressor is an electrical device that is powered by supply voltage. It has been designed so that no live parts are accessible. However, if ambient conditions are unfavourable or if the compressor or power cord is damaged, this protection may no longer be assured. Therefore, please follow the instructions below to avoid damage to the device and the associated danger of contact with live parts (e.g., electric shock):

- The compressor is not splashproof. Protect the device and the power cord from liquid and moisture. Never handle the device, the power cord or the power plug with damp hands. In addition, the device must not be operated in a damp environment.
- Only use the PARI power cord to operate the compressor.
- Never leave the compressor unattended while it is in use.

- For safety reasons, always disconnect the power plug from the socket under the following circumstances:
  - if a malfunction occurs during operation
  - before cleaning and maintaining the device
  - immediately after use
- Always pull the power plug, not the cable, to disconnect the power cord from the socket.
- Make sure that the power cord is never kinked, pinched or jammed. Do not pull the power cord over sharp edges.
- Keep the compressor and the power cord away from hot surfaces (e.g., stove top, electric fire, open fire). Direct heat may damage the compressor housing or the power cord insulation.
- Keep the power cord away from domestic animals (e.g., rodents). They may damage the power cord insulation.
- The compressor must not be operated and/or the power plug must be unplugged from the socket immediately if the compressor or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock.

## **1.6 Using the nebuliser**

Observe the following hygiene instructions:

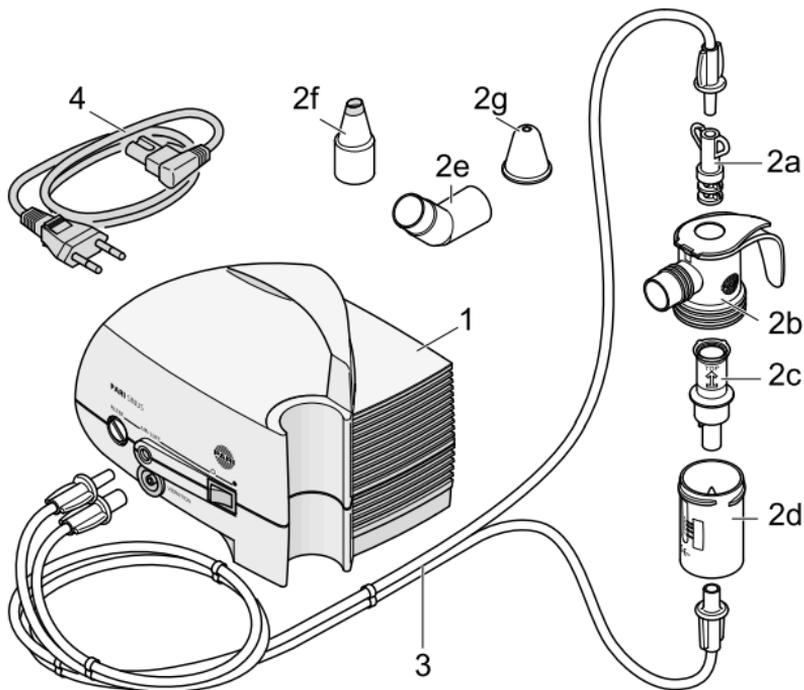
- Do not use product components unless they have been thoroughly cleaned and dried. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection.
- Before every use and cleaning, wash your hands thoroughly.
- Make absolutely sure you also carry out cleaning and drying before using the device for the first time.
- For cleaning and disinfection, always use fresh water with reduced calcium content.

- Make sure all components are dried properly after cleaning or disinfection.
- Do not keep the product and accessories in a damp environment or together with damp objects.

## 2 PRODUCT DESCRIPTION

### 2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.

(1)	PARI SINUS compressor incl. air filter
(2)	PARI LC SPRINT SINUS nebuliser, consisting of:
(2a)	Adapter (for vibration)
(2b)	Nebuliser upper section
(2c)	Nozzle attachment
(2d)	Nebuliser lower section
(2e)	Bend
(2f)	Nasal joining piece
(2g)	Nasal plug
(3)	PARI SINUS tube system, consisting of two identical tubes for compressed air and vibration
(4)	Power cord

## 2.2 Intended use

The PARI SINUS is a medical device intended for use in treating the upper airways with nebuliser therapy. It consists of a PARI SINUS compressor and a PARI LC SPRINT SINUS nebuliser.

The product is designed to treat patients from age 6 up to adult.

## 2.3 Proper use

The product is intended exclusively for home use.

The compressor is intended exclusively for indoor use.

For reasons of hygiene, the nebuliser, the nose plug and the nasal joining piece must only be used by a single patient.

This product is only designed for patients who are able to breathe by themselves and are conscious.

Saline solutions can be nebulised with the PARI SINUS to mobilise secretion. In this case, please follow the information in the instructions for use of the saline solution concerned.

If infection or inflammation of the paranasal sinuses is suspected, an appointment should be made with a doctor who can prescribe the medications suitable for use in nebuliser therapy with the PARI SINUS.

It is imperative that patients with cystic fibrosis discuss their treatment beforehand with a doctor.

An application usually takes 5 to 10 minutes.

## 2.4 Contraindications

Patients who respond to the treatment with adverse reactions must stop the therapy.

Following surgery or where inflammation of the middle ear is present, a risk/benefit assessment for vibrating aerosols must be carried out by the doctor before the start of treatment.

## 2.5 Material information

Polypropylene	Nebuliser, bend, adapter, nasal joining piece
Silicone	Nasal plug
Thermoplastic elastomer	Nebuliser

## 2.6 Product combinations

The PARI SINUS compressor can be used together with the following nebulisers:

- PARI LC SPRINT SINUS or PARI LC SPRINT XLent, for treatment of the upper airways with medication aerosols
- PARI LC Plus or PARI LC SPRINT, for treatment of the lower airways with medication aerosols

## 2.7 Operating life

Component	Expected life cycle
Compressor	5 years, equivalent to approx. 1,000 operating hours
Air filter	200 operating hours, max. 1 year
Nebuliser and PARI SINUS tube system	1 year

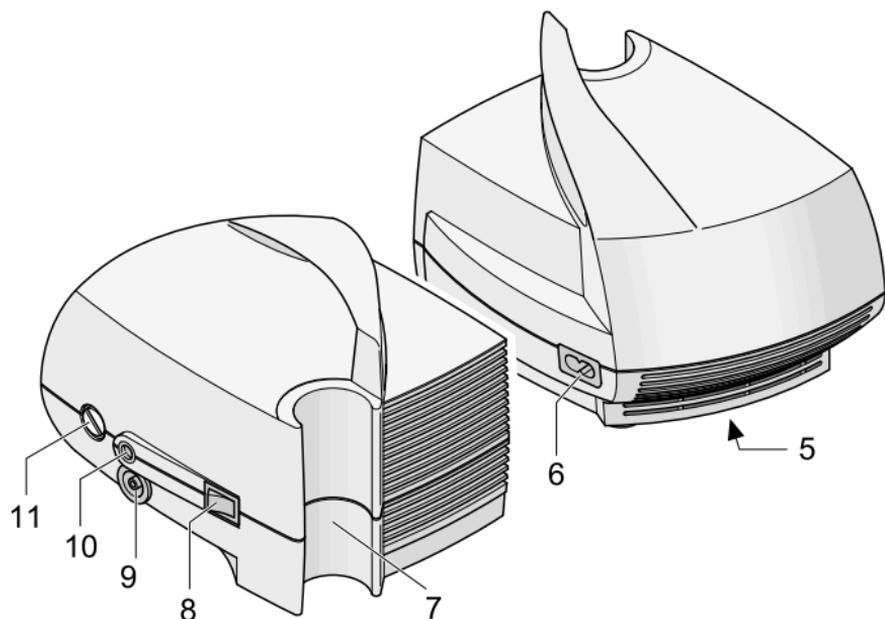
The nebuliser is designed for 365 applications and 52 disinfections.

If the treatment of a disease necessitates more intensive use and/or more frequent disinfections, the life cycle of the device is shortened.

The individual components must be replaced as soon as the expected life cycles indicated above have elapsed, if not sooner. A Year Pack for replacing the nebuliser, and also containing a new air filter, is available for purchase.

## 2.8 Working parts

The compressor includes the following working parts:

- (5) Type identification plate (bottom of device)
- (6) Power cord socket (rear of the device)
- (7) Holder for nebuliser
- (8) On/off switch
- (9) Vibration connection
- (10) Air connection
- (11) Filter holder

## **2.9 Mode of action in treatment of the upper airways**

Nebuliser treatment of acute and chronic inflammations of the paranasal sinuses (sinusitis) using inhaled medications (saline solutions and/or drugs) is subject to particular requirements. The aim is to apply the active agent through the ostia (connections between the main nasal cavity and paranasal sinuses) directly to its site of action, the nasal sinuses. To achieve this aim, the medical device creates a pulsating mist ("aerosol").<sup>1</sup> The pulsation helps the active agent to pass through the ostia and into the paranasal sinuses much more effectively. The advantage of such topical (local) therapy over oral, systemic application (e.g., in tablet form) is that doses can be smaller. Thus it can be expected that there is a reduction in the occurrence of side effects. It also moistens the mucous membranes of the nose.

## **3 TREATMENT**

### **3.1 Preparing the treatment**

#### **Setting up the compressor**

Check the compressor housing, the power cord and the power plug for damage every time before use. Make sure that the local supply voltage complies with the voltage data marked on the compressor identification label. Only use the compressor if all components are intact. The compressor must not be operated if the device or the power cord is damaged, or if damage is suspected (e.g., after it has fallen or if there is a smell of burning plastic).

---

1) Aerosol: Small particles of solid, liquid or mixed composition suspended in gases or air.

The room in which the compressor is operated must satisfy certain conditions [see: Ambient conditions for operation, page 76]. The compressor is not intended for outdoor use.

### CAUTION

When setting up the compressor, follow the instructions below to avoid any **risk of injury** or **damage to the device**:

- Place the compressor on a firm, flat, dust-free, dry surface. A soft base such as a sofa, a bed or tablecloth is not suitable because the compressor might fall.
- Route the power cord in such a way that it is not a tripping hazard and no one can become entangled in it.
- Make sure that children cannot pull on the power cord or tubing and cause the compressor to fall.
- Only operate the compressor in a low-dust environment. If it is operated in a very dusty environment (e.g., under the bed or in workshops), or on the floor, a larger amount of dust is likely to collect inside the housing, which may impair the quality of the treatment or damage the compressor.

### WARNING

To avoid a **risk of fire** due to a **short circuit**, follow the instructions below when setting up the compressor:

- Do not operate the compressor close to flammable objects such as curtains, tablecloths, or paper.
- Do not operate the compressor in areas where there is a risk of explosion or in the presence of oxidising gases (e.g., oxygen, nitrous oxide, flammable anaesthetics).

**⚠ WARNING**

**Liquids** can conduct electricity, thereby posing a **risk of electric shock**. Therefore, never take hold of the power cord or the power plug with damp or wet hands.

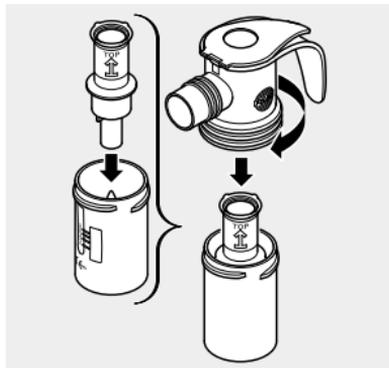
- Connect the power cord to the power cord socket of the compressor.
- Plug the power plug into a suitable socket.

## Assembling the nebuliser


**⚠ CAUTION**

Check all nebuliser components and the accessories before each use. Replace any broken, misshapen or seriously discoloured parts. Please also follow the instructions for assembly given below. Damaged components and/or an incorrectly assembled nebuliser may impair **functioning of the nebuliser** and thus **treatment as well**.


- Gently push the nozzle attachment as far as it will go onto the nozzle in the nebuliser lower section. The arrow on the nozzle attachment must point upwards.
- Place the nebuliser upper section on the nebuliser lower section and turn it clockwise to lock the two parts together.

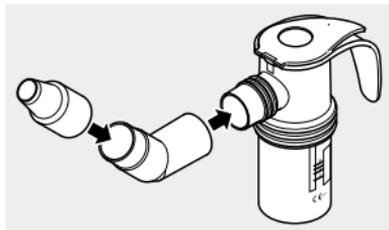- Connect the nasal joining piece to the bend and then connect both to the nebuliser.

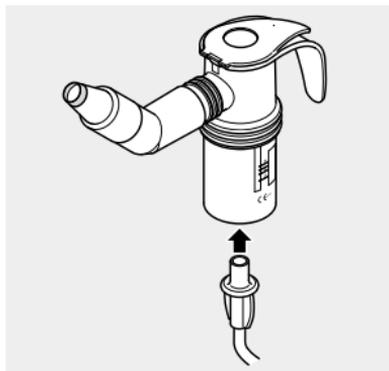- Attach the corresponding end of a tube to the underside of the nebuliser.


## Filling with medication

### NOTICE

Take care to ensure that the cap only moves in the direction allowed by the hinge. Otherwise the **nebuliser cap might snap off**.

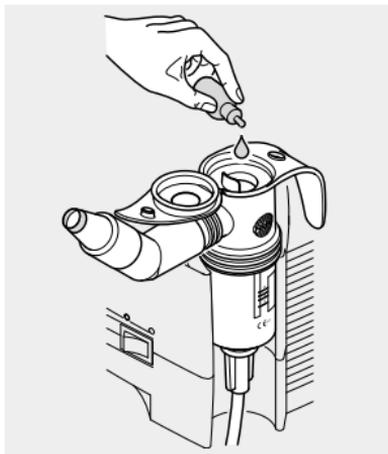- Release the closure on the nebuliser upper section by pressing your thumb against the underside of the cap.
- Insert the nebuliser in the holder on the compressor intended for this purpose.
- Pour the medication into the top of the nebuliser chamber. To find out the quantity to be used, please refer to the information in the instructions for use of the medication.

**i** *Ensure that the medication is filled no higher than the upper scale marking (max. level 8 ml). Overfilling can greatly impair the nebulisation of medication.*


- Close the cap of the nebuliser. Make sure that the cap snaps into place.

If you have to nebulise several medications one after the other in each application, rinse the nebuliser with tap water before filling with the next medication. Remove excess water by shaking the nebuliser and then fill it with the next medication as described.


## Connecting the tubes

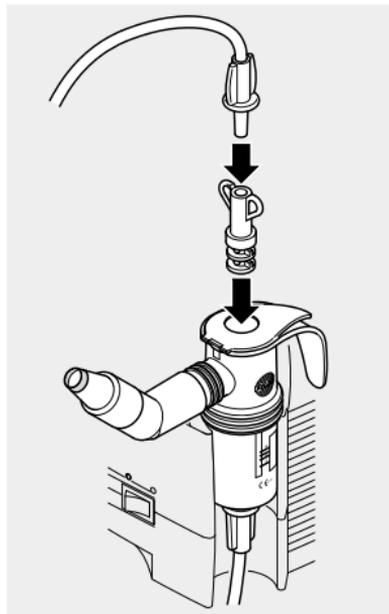The following section describes how to connect the PARI LC SPRINT SINUS nebuliser to the compressor. If you are using a different nebuliser, the adapter is not required, and only one of the two tubes is used for the compressed air supply. Then, the vibration connection on the compressor is left free.

- Insert the corresponding end of the second tube in the adapter.
- Insert the adapter in the nebuliser upper section.

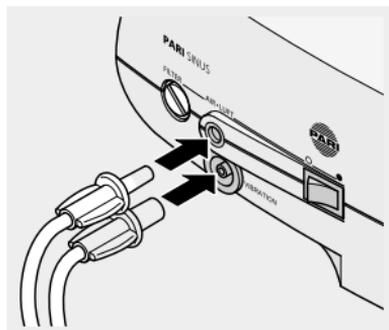- Insert the free ends of both tubes in the corresponding connections on the compressor. The tube ends can be inserted more easily if they are twisted slightly when inserting.

## 3.2 Performing treatment

### WARNING

In the event of **damage to the device** a person may come into contact with live parts, and there may be a **risk of electric shock**. Therefore, switch the compressor off and disconnect the power plug from the mains socket immediately if the compressor or the power cord is damaged or if damage is suspected (e.g., following a fall or if there is a smell of burning plastic).

Before starting the treatment, ensure that all parts are firmly connected to each other.

Perform the treatment until the aerosol is used up (indicated by a change in the sound of the nebuliser).

**i** *Several drops of medication will remain in the nebuliser after the treatment has ended.*

To ensure that the treatment has the desired effect, please follow the instructions below:

- In order for the aerosol to reach all regions of the paranasal sinuses, the nose must be clear. If blocked, rinse it beforehand using for example the PARI MONTESOL nasal douche. If necessary, use a product designed to reduce swelling.
- The aerosol that is generated during nebulisation should only be delivered to the paranasal sinuses. To ensure this is the case, the soft palate must be closed.<sup>2</sup>

**Info:** *If the soft palate is not closed properly, some of the aerosol will end up in your throat. One sign of this is that the*

---

2) The soft palate is the extension of the hard palate. It separates part of the oral cavity from the throat and is responsible for closing the airway off from the oesophagus.

*nostrils do not vibrate. In this case, it is possible that not enough saline solution or medication is reaching the paranasal sinuses.*

- The aerosol is automatically delivered to the paranasal sinuses. Therefore, you should hold your breath during the treatment. In order to breathe, pause the treatment briefly. Switch the compressor off during breathing pauses.

To close the soft palate, follow the instructions below:

To close your soft palate, make the sound "ing" (pronounced like the second syllable in Peking) for as long as possible. After a short breathing pause, repeat this as often as necessary until the treatment is complete.

**i** *The nose is automatically blocked off from the mouth and throat by closing the "soft palate". The soft palate closes when swallowing or making sounds such as "ing".*

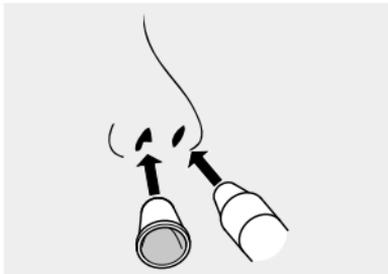
Alternatively, you can breathe in and out slowly and deliberately through your mouth, as if attempting to blow a candle out (i.e. do not breathe through your nose). However, this method of closing the soft palate is not easy to perform for everybody, and some practice may be required.

You can tell the soft pallet is closed when the nostrils vibrate gently and air comes out through the hole in the nasal plug in the unobstructed nostril.

To carry out the treatment with the PARI SINUS, proceed as follows:

- Sit in an upright position and relax.
- Take the nebuliser out of its holder and hold it upright.

- Block one nostril with the aid of the nasal plug supplied with the equipment.  
It is necessary to block one nostril with the nasal plug because a certain amount of resistance is needed to ensure that the vibrating aerosol can penetrate the paranasal sinuses.

**i** *The nasal plug has a small opening that creates a defined resistance when breathing.*

*Make sure that the nostril is not completely blocked (e.g. by pressing closed with a finger) because this could create too much resistance.*

- Position the nasal joining piece of the nebuliser against the free nostril so that the nostril is closed tightly.
- Close your soft palate (as described).
- Switch the compressor on.

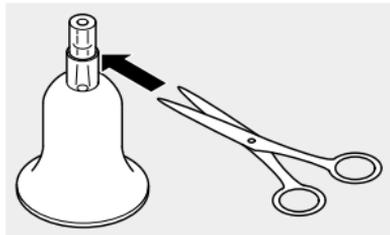
The aerosol is automatically conveyed into the paranasal sinuses.

Make sure that the nostrils vibrate gently for the entire period of treatment.

After about 2.5 minutes, carry out the treatment as described for a further 2.5 minutes on the other nostril. The treatment is complete after 5 minutes in all.

If you experience an unpleasant sensation of pressure in the ears - which can arise during the treatment - make the hole in the nasal plug larger (or do not press the free nostril closed so firmly):

- Push out the inside of the nasal plug and cut off the tip at the edge indicated using a pair of scissors.

**i** *The larger the hole, the less the resistance. Take extra care not to make the hole too large, since this will mean that resistance is too low during breathing. The vibration must be felt at all times during the treatment session.*

- Push the inside of the nasal plug back in.

### **3.3 Ending the treatment**

- Switch the compressor off.
- Place the nebuliser back in its holder.
- Disconnect the power plug from the mains socket.

**i** *Complete disconnection from the mains is only certain when the power plug has been unplugged from the socket.*

## 4 CARE

### 4.1 Preparation

- Pull the two tubes and the adapter off the nebuliser.
- Detach the adapter from the tube.
- Open the cap of the nebuliser.
- Make sure that all medication residues are removed from the nebuliser.
- Dismantle the nebuliser into its individual parts.
- Rinse all components of the nebuliser under running tap water. This must be done especially thoroughly if cleaning and disinfection are not carried out straight away.

### 4.2 Drying the connection tubes

**i** *The PARI SINUS tube system must not be cleaned or disinfected.*

- Insert one of the two tubes into the air connection on the compressor.
- Switch the compressor on.
- Allow air to blow through the tube until the "Air" has removed all moisture from the tube.
- Switch the compressor off.
- Disconnect the tube from the compressor, and attach the other tube to the air connection.
- Repeat the process as described for the first tube.

## 4.3 Care of the compressor

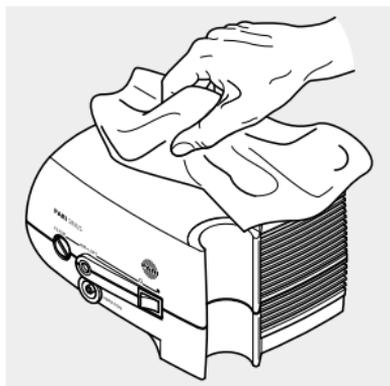
### Cleaning

#### **WARNING**

**Liquids** can conduct electricity, thereby posing a **risk of electric shock**. Therefore, always switch the compressor off and disconnect the power plug from the mains socket before starting to clean it.

- Wipe the outer surface of the housing with a clean, damp cloth.

#### **NOTICE**

**Liquids that get into the device** can cause a **fault in the device**. Therefore, never spray any liquids onto the compressor or the power cord.

If liquid does get into the compressor, the device must not be used under any circumstances. Before using the compressor again, you must contact the PARI GmbH Technical Service department or the PARI Service Center.

## Filter change

In normal operating conditions, the compressor filter must be replaced after every 200 operating hours, but at least once a year.

Also please check the filter at regular intervals (after every 10 applications). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new one if it has become damp.

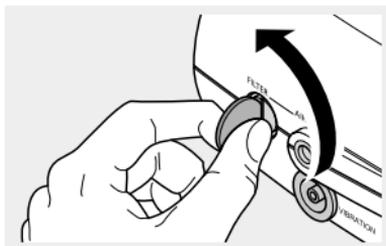
### The filter cannot be cleaned and then reused!

#### **NOTICE**

Only use PARI air filters that are intended for use with your compressor, otherwise your **compressor might be damaged**.

To replace the filter, proceed as follows:

- Unscrew the filter holder from your compressor using a suitable coin.
- Pull the old filter out of the filter holder and fit the new filter in its place.


- Reinsert the filter holder together with the new filter in your compressor and screw it tight with the coin.

## Servicing

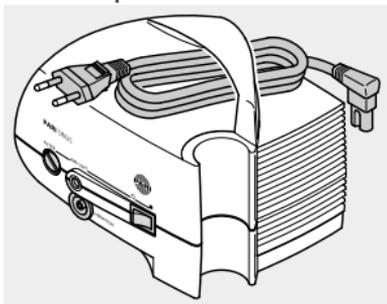The compressor must only be serviced by the PARI GmbH Technical Service department or a service centre expressly authorised by PARI. If the compressor is opened or manipulated by anyone else, all claims under the warranty shall be void. In these cases, PARI GmbH will also accept no liability.

## Storage

- Disconnect the power cord from the compressor.
- Wind up the power cord and stow it under the compressor handle.

### **NOTICE**

The power cord can be damaged if it is kinked. Therefore, do not wind the power cord around the compressor.

Store the compressor and accessories safely out of continuous direct sunlight.

Always disconnect the compressor from the mains for storage because electrical devices that remain plugged in present a potential hazard source.

## 4.4 Care of the nebuliser and accessories

The nebuliser and the accessories used must be cleaned thoroughly after each application and disinfected at least once a week. For high risk groups (e.g., cystic fibrosis patients), the nebuliser and the accessories used must be disinfected at least once a day.

### Cleaning

- Place all components (except the PARI SINUS tube system) in warm tap water and a little dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose residue (the brush must be reserved exclusively for this purpose).
- Rinse all parts thoroughly in running water.
- You can remove excess water more quickly by shaking all parts.

## Disinfection

After cleaning, disinfect the dismantled nebuliser and accessories used (except the PARI SINUS tube system). Effective disinfection is only possible if the nebuliser has been cleaned.

### CAUTION

A damp environment may encourage the growth of bacteria. Therefore, remove all parts from the pot or disinfectant as soon as disinfection has finished. Dry the parts. The **risk of infection** is reduced when the parts are dried completely.

### *In boiling water*

- Place all the individual parts in boiling water for at least 5 minutes.  
Use a clean pot and fresh water with reduced calcium content, if possible.

**i** *Plastic will melt if it comes into contact with the hot base of the pot. Therefore, make sure there is plenty of water in the pot. This way you will avoid damaging the components.*

- You can remove excess water more quickly by shaking all parts.

## *Using a standard thermal disinfecter for baby bottles (not a microwave oven)*

For effective disinfection, use a disinfecter with a runtime of at least 6 minutes. Regarding disinfection, the duration of the disinfection procedure and the quantity of water required for this, follow the instructions for use of the disinfecter you are using.

### **CAUTION**

Inadequate disinfection encourages the growth of bacteria and thus increases the **risk of infection**. Thorough disinfection has not been completed until the disinfecter automatically switches itself off, or the minimum disinfection time specified in the instructions for use of the disinfecter has elapsed. Therefore, do not switch the device off prematurely. Also make sure that the disinfecter is kept clean and regularly check that it is in good working order.

## *With a microwave disinfecter*

Microwave disinfecters ensure effective disinfection of the product components in a standard commercial microwave oven.

Place the product components in the disinfecter with enough space between them to allow the steam to reach all surfaces reliably.

Maintain the following times when disinfecting:

Wattage	Disinfection time	Cooling time
850 – 950	4 min.	2 min.
600 – 850	6 min.	2 min.

Information about the amount of water required and about the disinfection process is included in the instructions for use of the devices in question (disinfector and microwave).

 **CAUTION**

The product components must not be exposed to microwave radiation unless they are in a microwave disinfector. If they are heated in a microwave oven without the protection of the disinfector, the **individual parts will be damaged**.

Do not use the "grill" function because this too will **damage the individual parts**.

At the end of the disinfection process, the parts may still be very hot. Therefore, it is essential to wait for the cooling time to pass before taking the parts out, otherwise there is a **risk of burns**.

Inadequate or ineffective disinfection encourages the growth of bacteria and thus increases the **risk of infection**. Disinfection is not completed properly unless both the specified minimum disinfection time and the required power output of the microwave are observed. The disinfector must also be perfectly clean. Check regularly that the microwave and the disinfector are functioning perfectly.

Use only water for disinfection, never use chemicals.

## Chemical cleaning with disinfection

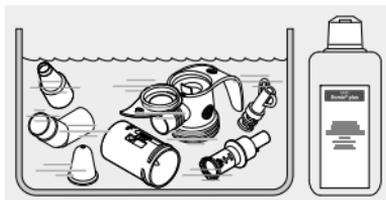
Cleaning and disinfection can be carried out in a single cycle using a chemical preparation process. For this, use the cleaning disinfecting agent Bomix® plus.

To ensure safety when handling chemicals, follow the instructions for use of the disinfecting agent, particularly the accompanying safety instructions.

### CAUTION

Inadequate disinfection encourages the growth of bacteria and thus increases the **risk of infection**. Adequate cleaning with disinfection can only be assured if the specified mixing ratio and application time are adhered to, and if all individual parts are completely immersed in the solution for the entire application time. There must not be any air pockets or bubbles.

- Prepare a 2% solution of Bomix® plus by mixing 10 ml of the concentrate with 500 ml tap water.
- Place all the individual parts in the prepared solution and leave them to soak for 5 min.

**i** *If the application period is exceeded significantly, the plastic parts may take on the smell of the disinfectant.*

- Rinse off all parts thoroughly in running water (residues of the disinfectant can cause allergic reactions or irritations of the mucous membrane).
- You can remove excess water more quickly by shaking all parts.
- Dispose of the used solution (the diluted solution can be got rid of down the drain).

## Drying and storage

- Place all parts on a dry, clean and absorbent surface and let them dry completely.
- Wrap the nebuliser in a clean, lint-free cloth (e.g., a tea-towel) and keep it in a dry, dust-free environment.

## 5 TROUBLESHOOTING

Fault	Possible cause	Procedure
The compressor does not start up.	The power cord is not plugged into the device properly.	Check that the power cord is seated correctly in the compressor power cord socket.
	The power plug is not plugged into the mains socket properly.	Check that the power plug is seated correctly in the mains socket.
No medication is coming out of the nebuliser.	The nebuliser nozzle is blocked.	Clean the nebuliser.
	The compressed air supply tube is not connected to the air connection on the compressor correctly.	Check that both ends of the tubing are seated properly.
	The tube attached to the air connection on the compressor is leaking.	Replace the defective tube or the entire PARI SINUS tube system.

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center.

## 6 TECHNICAL DATA

### 6.1 General compressor data

Compressor type	Double compressor: Piston for generation of compressed air Diaphragm for generation of bolus
Supply voltage [V] Mains frequency [Hz] Power consumption [A]	see back cover page
Max. compressor flow (FreeFlow)	9.5 l/min.
Compressor flow <sup>1)</sup>	4.8 l/min.
Sound pressure level <sup>2)</sup>	approx. 56 dBA
Operating pressure <sup>3)</sup>	150 kPa (1.5 bar)
Bolus pressure amplitude, absolute	20 kPa (0.2 bar)
Frequency of bolus pressure amplitude	44 Hz
Housing dimensions [cm]	19.2 × 14.5 × 15.0 (W × H × D)
Weight	1.8 kg
Flow	4.5 l/min.

1) Towards PARI nebuliser nozzle (Ø 0.48 mm)

2) Maximum A-evaluated sound pressure level of the compressor (in accordance with DIN EN 13544-1, section 26)

3) With LC SPRINT SINUS nebuliser

## 6.2 Nebuliser data

Operating gases	Air, oxygen
Minimum flow	3.0 l/min.
Corresponding minimum operating pressure	0.5 bar / 50 kPa
Maximum flow	6.0 l/min.
Corresponding maximum operating pressure	2.0 bar / 200 kPa
Minimum fill volume	2 ml
Maximum fill volume	8 ml

Aerosol data according to DIN EN 3544-1 is available on request from PARI GmbH or on the internet at [www.pari.de/de-en/products](http://www.pari.de/de-en/products) (under "Technical data" on the respective product page).

## 6.3 Classification according to DIN EN 60601-1

Type of electric shock protection	Protection class II
Degree of protection from electric shock of the part used (nebuliser)	Type BF
Degree of protection against water ingress in accordance with EN 60529 (IP rating)	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air, with oxygen, or with nitrous oxide	No protection
Operating mode	Continuous operation

## **6.4 Information about electromagnetic compatibility**

Electrical medical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Such equipment must only be installed and operated in accordance with EMC instructions.

Portable and mobile high-frequency communication devices can disrupt electrical medical equipment. Using accessories, converters and power cords other than those specified (with the exception of converters and power cords that the manufacturer of the medical electrical device sells as spare parts for internal components) can result in higher emission levels or lower the resistance to interference of the device.

The device must not be placed directly beside or on top of other devices for operation. If the medical electrical device must be placed beside or on top of other devices to operate it, it should be monitored constantly to ensure that it is operating properly in the arrangement used.

Technical data on electromagnetic compatibility (EMC instructions) in table format are included with the device. These can also be ordered from PARI GmbH or downloaded from the internet at [www.pari.de/de-en/products](http://www.pari.de/de-en/products) under "Technical Data" on the respective product page.

## **6.5 Recommended safety distances**

Recommended safety distances between portable and mobile high-frequency (HF) telecommunication devices and the device:

The product is intended for operation in an electromagnetic environment in which radiated HF transients are monitored. The user can help to prevent electromagnetic interference by observing the minimum safety distances between portable and

mobile HF communication equipment (transmitters) and the product, as recommended below in accordance with the maximum output power of the communication device.

Safety distance depending on transmitting frequency:

Rated power of the transmitter	150 kHz to 80 MHz $d = (3.5/3) \times \text{SQRT}(P)$	80 MHz to 800 MHz $d = (3.5/3) \times \text{SQRT}(P)$	800 MHz to 2.5 GHz $d = (7/3) \times \text{SQRT}(P)$
0.01 W	0.2 m	0.2 m	0.3 m
0.1 W	0.4 m	0.4 m	0.8 m
1 W	1.2 m	1.2 m	2.3 m
10 W	3.8 m	3.8 m	7.3 m
100 W	12 m	12 m	23 m

For transmitters whose rated power does not appear in the table above, the safety distance can be calculated using the equation associated with the respective column, where "P" is the rated power of the transmitter in Watt (W) according to the information supplied by the device manufacturer.

## 6.6 Ambient conditions for operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 hPa

## 7 MISCELLANEOUS

### 7.1 Transportation and storage

Minimum ambient temperature (without monitoring of relative humidity)	-25 °C
Maximum ambient temperature (with relative humidity of up to 93%)	+70 °C
Humidity	max. 93%
Atmospheric pressure	500 hPa – 1,060 hPa

### 7.2 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE).<sup>3</sup> Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers). Materials recycling helps to reduce the consumption of raw materials and protect the environment.

### 7.3 Terms and conditions of warranty

PARI guarantees that your device, if used according to the instructions, will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate, beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided by PARI applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects are not limited by the warranty or any

<sup>3</sup>) WEEE = Directive 2002/96/EC of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of January 27, 2003 on waste electrical and electronic equipment.

claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

### **What does the warranty cover?**

If, exceptionally, a defect is discovered, PARI will at its discretion repair or replace the device, or refund the purchase price of the product. If it is replaced, the replacement device may either be the same model or a model that is at least comparably equipped. Replacement or repair of the device shall not serve as the basis for a new warranty. All replaced old devices or parts shall become the property of PARI. Further claims are excluded. This applies particularly for any claims for compensatory damages. This disclaimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful wrongdoing and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

### **The warranty shall be cancelled if**

- the device has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, etc.
- the damage was caused by transporting the device incorrectly or a falling impact
- the device has been misused or not cared for correctly
- the serial number on the device has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the device by persons not authorised by PARI

Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the entire device to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

## 7.4 Explanation of symbols

The following symbols can be found on the product and/or the packaging:

	Please follow the instructions for use.
	Manufacturer
	This product satisfies the requirements of 93/42/EEC (Medical devices) and 2011/65/EU (RoHS).
	Order no.
	Serial number of the device
	Lot no.
	Off / On
	Alternating current
	Protection class of the part used: Type BF
	Protect device from moisture
	Protection class II device
	Minimum and maximum ambient temperature
	Minimum and maximum humidity

	Minimum and maximum air pressure
	The medical device was distributed commercially after 13 August 2005. <b>This product must not be disposed of with normal domestic waste.</b> The symbol of the refuse bin with a cross through it indicates that it must be disposed of separately.

## 7.5 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)

+49 (0)8151-279 220 (international)

# **PARI SINUS**

## **CERTIFICATE OF GUARANTEE**

We guarantee this device for 3 years,  
commencing on the date of purchase.

PARI GmbH  
Technical Service Dept.  
Holzhofstr. 10b  
82362 Weilheim, Germany

Application no:

Bar code label

Confirmation of purchase:

The appliance with the above appliance number  
was sold in its original packaging.

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©2015 PARI GmbH Spezialisten für effektive Inhalation, 028D1020-A 08/15