## FFP2 Virus-Deactivating Respirator With Nanofiber Filter Inside

protection against  
viruses and bacteria

unique nanofiber  
filter inside

fully certified  
FFP2 respirator

1 pc = 7 days  
of protection

revolutionary accelerated  
copper technology

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EN

### Product Information

**VK RespiRaptor®** is a respirator classified as **FFP2 NR** which protects the user from the influence of his or her environment. It is fully certified in accordance with **EN 149:2001+A1:2009 standards** and its filtration efficiency also complies with requirements for respirators certified as N95, KN95 and GB2626-2006. The respirator is self-sterilizing and therefore prevents cross-contamination. The respirator works on a double barrier principle – its nanofiber membrane immediately captures viruses, bacteria, solid particles and aerosols (smog, pollen, allergens). The accelerated copper CAS 7440-50-8 subsequently ensures deactivation of captured viruses and bacteria. 100 g of product contains 0.6 g of copper (one size L/M respirator contains 0.023 g of copper). With regard to the FFP2 certification, the respirator is certified as NR – for a single use and for only one working shift. You can use the respirator for up to 7 days to stay protected against viruses and bacteria, and that is because the accelerated copper inside it deactivates viruses and bacteria incessantly and repeatedly.

**Composition:** treated object. **Use of biocidal agent:** PT2 respirator.

**Category of users:** general public. **CE Notified body:** Géptest Ltd., Gyenes utca 12., 1032 Budapest, Hungary, notified body 2233.

**Storage:** store unpacked in the original packaging in a dry and dark place at the minimum storage temperature of -10 °C, the maximum storage temperature +40 °C, and the maximum air humidity of 80%.

**Storage conditions:** 5 years from the date of manufacture (when stored in an undamaged packaging).

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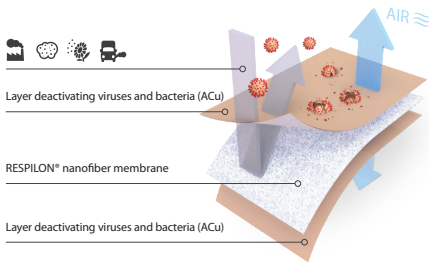
### RESPILON® VK Membrane

**Filter:** RESPILON® nanofiber membrane captures viruses, bacteria, solid particles and aerosols (smog, pollen, allergens).  
**Before you use the respirator:** Always check the condition of the respirator before you use it. Check it for any defects, signs of suspiciously looking stains or any signs of damage. Do not use the respirator in case of mechanical damage.

EU declaration of conformity [www.respilon.com](http://www.respilon.com)

Manufactured by RESPILON in EU. Designed by RESPILON in EU

### Warning

Do not use in an environment with a lower oxygen concentration. If there is hair or facial hair on the seal line of the respirator the quality of the face-fit may be compromised. No direct or indirect adverse side effects of accelerated copper CAS.7440-50-8 are known. If it is causing you any discomfort, breathing complications, or a rash, etc., stop using the respirator as soon as possible.

### How to put on the respirator properly

1. Sanitize your hands before opening the package, take out the respirator and unfold it.
2. Place the respirator on the root of your nose and hold it with one hand.
3. Use your other hand to pull the ear loops behind your ears.
4. Press the nose clip firmly around the root of your nose.
5. Join both ear loops with the plastic hook over the back of your head.

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